# Vivendi

NON-ALDEHYDE BIO-PATCH

# Vivendi

*L-Hydro Pericardium Patch*

Labcor Laboratórios is pleased to introduce the latest advancement in the preservation process of its new **Vivendi** bovine pericardium patch. **Vivendi** is treated with its latest development in tissue treatment known as **L-Hydro** technology.

## CLINICAL BENEFITS

- ◆ Non-thrombogenic surface
- ◆ Reduced antigenicity and cytotoxicity
- ◆ Soft, pliable tissue (Native-like tissue)
- ◆ Easy handling
- ◆ Reduced adhesion formation
- ◆ Low Transudation
- ◆ Nominal calcification
- ◆ Non-fraying pericardium
- ◆ Diminished suture hole bleeding

## SIZES OF THE PATCH

- ◆ 2cmx5cm, 10cmx2cm, 10cmx5cm, 10cmx10cm, 10cmx18cm, 10cmx26cm

## L-Hydro Technology

*New generation for tissue material with non-aldehyde preservation*

Vivendi is part of the new generation of Labor products, preserved by the L-Hydro technology. This new preservation process presents to the receiving organism a non-viable matrix very similar to nature. Moreover, it permits cellular growth to occur in the condition under which tissue would exercise its functions.

## PRESERVATION PROCESS

- ◆ Mild extraction of antigenic substances;
- ◆ Non-extracted antigens are hidden by means of an activated poly-glycol agent;
- ◆ Mild chemical process to promote the reduction of components that induce inflammatory responses on the receptor;
- ◆ Incorporation of a non-steroid anti-inflammatory agent;
- ◆ Incorporation of an anti-thrombosis agent;
- ◆ Chemical sterilization in hydrogen peroxide.

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Av. Silva Lovo, 1670 - Nova Granada - Belo Horizonte/MG - Brasil \* CEP 30431-262

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